




RECOMMENDED SENSOR CARE

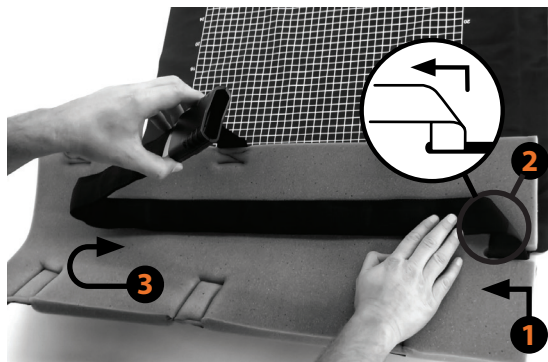

- 1 Slide the foam carrier's slit around the cable.
- 2 Loosely fold the cable at a right angle.
- 3 Fold the remaining length of cable back towards the slit, with the connector facing the slit.

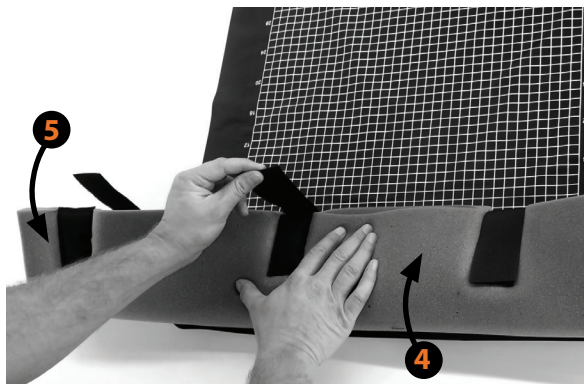

- 4 Fold the foam carrier around the cable.
- 5 Secure the Velcro straps.

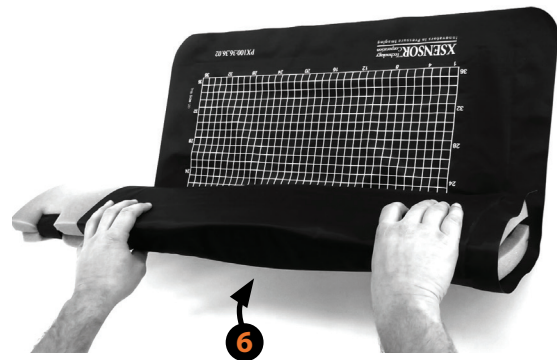

- 6 Roll the sensor around the foam Carrier.

WARNING



Folding the sensor in half may permanently damage the sensor.



Gripping, bunching up, and/or tugging on the sensor may permanently damage the sensor.



Storing the sensor inside the foam carrier will damage the sensor.